

SHIPPING & HANDLING

AVIAN INFLUENZA BHI SAMPLES


Place BHI tube(s) in foam protective sleeve(s) and secure in labeled Whirlpak Bag. Be sure caps are **tightened** to prevent leakage.



Place both Whirlpak bags into a larger zip-lock sample bag.



In a separate Whirlpak bag, enclose the completed submission form.

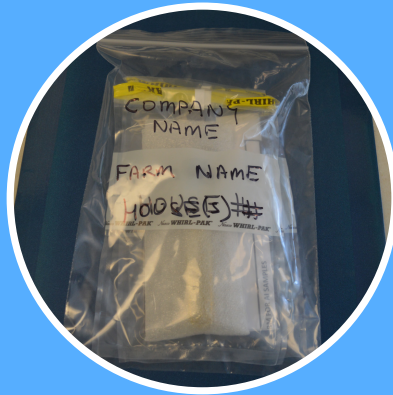

Wrap sample bag in bubble wrap or similar material.


Place ice pack on bottom of shipping cooler



Fill void areas with packing material to prevent contents from moving during shipment.



Place bubbled wrapped sample bag on bottom ice pack & add a second ice pack on top

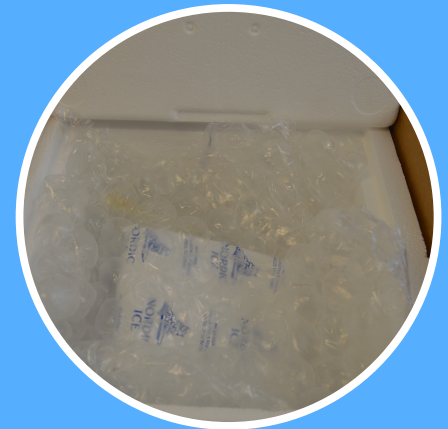

Ship Overnight. Check **Customer Portal** the next day to confirm receipt.